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1. Technical details and scope of supply

Item no.: PM-EVA-7053-H
suitable for all bed types, mattresses and bed frames
Color: white
Dimensions: 2.000 x 870 mm
for bed size: length 190 to 205 cm
width 80 to 100 cm
ILN: 4250355370532
Material: PVC coated polyester fabric,
with ventilation openings,
latex free

Scope of supply:

- Evacuation sheet with side wings
incl. control tag and four rubber strips for fixing
- Manual

**German TUV approved according to
DIN EN 1865:2010**

Made in Germany

2. Use and operation

The evacuation system was exclusively designed for the use in rehabilitation beds for the quick, safe and gentle evacuation of non ambulant persons. Place the evacuation sheet in the middle of the bed directly on the slatted frame. The folded belts with belt buckle are positioned with a rubber band each. The plugs of the belt buckles should be positioned at the accessible side of the bed.

The mattress must be put on the evacuation sheet and the bulk belts should be covered by the mattress. Mark each bed that contains an evacuation sheet by leaving the control tag visibly outside.

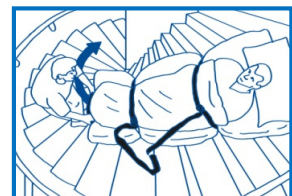
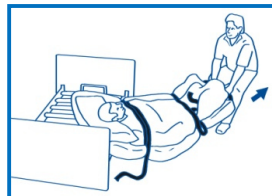
Option:

If necessary you can fix the evacuation sheet to the mattress with the four rubber strips enclosed. Draw the rubber strip each through the perforation opening that is nearest to each edge of the evacuation sheet. The strip is like a loop and can be slipped over each edge of the mattress. Now the evacuation sheet is fixed to the mattress.

3. Rescue situation

Stand in front of the accessible side of the bed. If possible, lower the head and leg part of the bed into the lying position. Put the bedding carefully on and around the patient. Pull both plugs of the belts out from under the mattress. Knee on the bed and pull the opposite belt parts out from under the mattress. Click into place both parts of the belts, each with its opposite counterpart.

Pull each of the belts with its loose end strongly and as much as possible. This way you will obtain an u-form of the mattress that could protect the person lying in it from crushes. The mattress is fixed safely on the evacuation sheet now. With taking the large handles of the evacuation sheet at the head or leg side you can pull the sheet together with the fixed mattress out from the bed over the floor, through doors and/or over stairs to an area which is safe for the patient. If the rescue situation makes it necessary (to get over an obstacle for example) you can carry the evacuation sheet for a short time by using the handles on both sides, head or leg side of the sheet. There are additional carry handles in the middle of the long side of the sheet that allows carrying the evacuation sheet with up to 6 assistants.

4. Maintenance / Care

The evacuation sheet does not need special servicing or maintaining. Take a household cleaner and clean the evacuation sheet regularly with a damp cloth by hygienic reasons.

If necessary, the evacuation sheet can be treated with a usual wipe disinfectant (e.g. ethanol-based). A principal recommendation or respectively an exclusion of disinfectants cannot be given due to the wide range of products. Please check the compatibility of the material before first use. We will be glad to send you a sample piece of material for testing purposes. Please note that the use of any underlays in beds as also evacuation sheets requires a certain hygiene (see also § 36 protection against infection, hygienic schedule – prevention of mildew, accumulation of moistness). The evacuation sheet can be singly autoclaved up to 70°C (max.).

WARNING: The evacuation sheet is not suitable for the use in tumblers.

Do not use abrasive chlorine-based washing agents!

Dry the sheet thoroughly after treatment!

5. Life Time

If the evacuation sheet is stored appropriately it can be used for 20 years (rescue situation excluded).

6. Guarantee

- Petermann warrants that the evacuation sheet is free from defects in material and workmanship.
- This warranty shall remain in effect for 20 years.
- The warranty does **not** cover any failure of the unit during the warranty period if the failure is not caused by a defect in material or workmanship or if the failure is caused by unreasonable use or faults caused by a rescue situation, fire etc.
- Petermann is not liable for incidents or consequential injury or damage to property.

7. Safety Instructions

WARNING:

- **Evacuation sheets that are once used for an evacuation must be disposed of!**
- **The evacuation sheet is not suitable for the use in tumblers.**
- **Do not use abrasive chlorine-based washing agents!**
- **Dry the sheet thoroughly after treatment!**

8. Product Labeling / CE

The Petermann evacuation system is a medical product class I and complies with the MDD directive 93/42/EEC.

The product is TUV tested and complies with the requirements of DIN EN 1865:2010.

Petermann GmbH is certified in accordance with the directive DIN EN ISO 13485 by the German TUV.

9. Disposal Information

This product has been delivered by an environmentally aware manufacturer. The evacuation sheet is designed for a long-time use or for rescue situations. If necessary dispose the product in accordance with your local and national directives.

If you have any questions about the use of our products, please call our service number:

+49 (0) 9868 9339-0

Petermann GmbH • Waldweg 3 • D-91601 Dombühl
Phone: +49 (0) 9868 9339-0 • Fax: +49 (0) 9868 9339-20
E-mail: petermann@pm-med.de • www.pm-med.de

"Information subject to technical modifications to our products and deviations in colour or printing errors"

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